

Error code No.	Recorder display	Possible cause
1.	E.001	Calculation error
2.	E.002	Strong shake. The patient has moved his arm, hand or fingers to strongly.
3.	E.003	Invalid values. The pressure difference between systolic and diastolic is less than 10 mm/Hg or no pulse is registered after an elapsed time of 5 seconds.
4.	E.004	Error in data communication
5.	BAT	Batteries are empty.
6.	E.006	No pressure can be build up in the cuff. The pressure tube has been separated from the cuff or other leaks occur.
7.	E.007	Memory is full. No more than 300 measurements can be stored.
8.	E.008	Memory is empty.
9.	E.009	Residue pressure in the cuff